HighPerform Arm

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EN - INSTRUCTIONS FOR USE

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IT - ISTRUZIONI D'USO

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DA BRUGSANVISNING

SV - BRUKSANVISNING

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NO BRUKSANVISNING

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ALL ABOUT YOUR HIGHPERFORM ARM SLEEVES

How to put on your HighPerform arm sleeves properly

Please put on your HighPerform arm sleeves in a comfortable and stable position.

- 1 Position the arm sleeve so you can see the logo on the back of your hand.
- 2 Pull the sleeve over your hand and all the way up to your elbow. There, you can see two particularly stretchable zones which should be positioned around your elbow and the crook of your elbow.
- 3 Now pull the upper part of the sleeve over the elbow and gradually towards the shoulder.
- 4 Spread out the material of the sleeve so that there are no creases. When the arm sleeve is in the right position, it will neither cause uncomfortable pressure or pinching, nor restrict your movements.

General information

Please make sure you read the following information before using your HighPerform Sleeves for the first time and keep it safe for future reference.

Please note:

The sleeves can only be fully effective when they have been put on correctly. Your sleeves are largely resistant to oils, ointments, skin moisture and other environmental influences. However, the durability of the knitted fabric may be affected by certain combinations of the above. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, the product is not suitable for use by more than one person. The sewn-in textile label is important for the identification and traceability of the

product. We therefore ask you not to remove the label under any circumstances.

Why should I wear the HighPerform?

In sports, fractions of a second determine a victory or loss. Those who react more quickly, have better control of their body, higher precision and more speed as well as explosive strength will have the crucial advantage at the end.

The HighPerform arm sleeves by Sportomedix can support you during sporting activities and provide the following advantages:

- = improved blood circulation and increased oxygen supply to the muscles
- = fewer muscle vibrations, micro injuries and aching muscles
- = delayed muscle fatigue
- = quicker and better recovery
- = improved proprioception and muscle co-ordination
- = measurable performance increase

Can be worn during training, competitions and, of course, when recovering.

What makes the HighPerform arm sleeves so special?

HighPerform combines innovative muscle compression and effective stimulation of your fascia trigger points. The anatomical pressure gradient and the perfectly synchronised knit zones support circulation, thus improving the oxygen supply to the muscles. This counteracts early fatigue during sporting activities and positively influences recovery. Our HighPerform knitted fabric is particularly breathable, resilient and robust. Thanks to the special, particularly stretchable zone around the elbow and another extra soft zone in the crook of the

elbow, the HighPerform arm sleeves ensure maximum freedom of movement, a unique fit and perfect comfort. This will help you to give your best with every shot and strike, every quick movement, during training or a competition.

What effect do the patent-pending F+ Trigger Zones have?

Right above the trigger points, special F+ Trigger Zones have been incorporated. They can stimulate your fascias and muscles and thus relieve muscle imbalances, shortening and hardening. Another positive effect: they can improve your proprioception and muscle co-ordination.

Side effects

There are no known side effects when this product is used correctly. If negative changes (e.g. skin irritation) should occur during use, please take off the product and contact our Customer Service Team at contact@sportomedix.com straight away. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

Material composition

Please see the sewn-in label for more specific details.

Washing and care instructions

We recommend you turn your HighPerform sleeves inside out before washing them. You should wash the HighPerform separately the first time because the colour may run. Please observe the instructions on the sewn-in label. When hand washing, please

rinse the sleeves well and leave them to dry – lying flat if possible – at room temperature. Do not wring them out. We recommend a mild laundry detergent for delicate fabrics. You can reduce the drying time by placing the sleeves on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave them in the towel and do not dry them on a radiator or in the sun. Please use the delicates setting when drying them in a tumble dryer.

Correct storage, shelf life and usage duration

Do not store your HighPerform with sharp or heavy objects. Store at room temperature in a dry place and protect from direct sunlight. The product has a two-year shelf life. The usage duration with normal use is one year.

Disposal

As a rule, the product should be destroyed via the conventional disposal route after its end of use. Please observe the applicable local/national regulations.

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Julius Zorn GmbH