***HighPerform***  
***Wade / Oberschenkel***  
***Calf / Upper Leg***

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## **EN All about your HighPerform**

### **Dressing instruction**

Please put on your HighPerform on and off while in a comfortable seated position.

- 1 Make sure that the logo is at the front. To make donning of your thigh support easier, fold the silicone border outwards.
- 2 Pull the sleeve over your calf / thigh.
- 3 Ensure the sleeve fits your body accurately.
- 4 The trigger point stimulation should be at the back on your muscle.

In order to increase the wearing comfort of your compression garments, you can hoist the silicone border a little (if there is one) a number of times during wearing. Caution: rough, unfilled finger nails or sharp edged rings etc. may damage the knitted fabric.

### **General information**

Please make sure you read the following information before using your HighPerform sleeves for the first time, and keep it safe for future reference.

### **Please note!**

The sleeves can only be fully effective when they have been put on correctly. Your sleeves are largely resistant to oils, ointments, skin moisture and other environmental influences. However, the durability of the knitted fabric may be affected by certain combinations of the above. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, the product is not suitable for use by more than one person. The sewn-in textile label is important for the identification and

traceability of the product. We therefore ask you not to remove the label under any circumstances.

### **Notes for use**

#### **This is how compression can help during sports:**

- Better blood circulation
- Prevention of cramps in the legs
- Prevention of muscle strains
- Less muscle soreness
- Faster recovery (e.g. after torn muscle fibers)
- Faster regeneration

### **Side effects**

There are no known side effects when this product is used correctly. If negative changes (e.g. skin irritation, swelling of unknown cause, open wounds) should occur during use, please take off the product and contact our Customer Service Team at [contact@sportomedix.com](mailto:contact@sportomedix.com) straight away. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

### **Material composition**

Please see the sewn-in label for more specific details.

### **Washing and care instructions**

We recommend you turn your HighPerform sleeves inside out before washing them and wash them at 40 °C. You should wash the HighPerform separately the first time because the colour may run. Please observe the instructions on the sewn-in label. When hand washing, please rinse the sleeves well and leave them to dry – lying flat if possible – at room temperature. Do not wring them out. We recommend a mild

laundry detergent for delicate fabrics. You can reduce the drying time by placing the sleeves on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave them in the towel and do not dry them on a radiator or in the sun. Please use the delicates setting when drying them in a tumble dryer.

### **Correct storage, shelf life and usage duration**

Do not store your HighPerform with sharp or heavy objects. Store at room temperature in a dry place and protect from direct sunlight. The product can be stored for a maximum of 36 months. With daily wearing and washing of the product will result in a maximum service life of 6 months follows. In the case of irregular use, the wearing time is extended accordingly, but up to a maximum of 12 months.

### **Disposal**

As a rule, the product should be destroyed via the conventional disposal route after its end of use. Please observe the applicable local/national regulations.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please send an email to [contact@sportomedix.com](mailto:contact@sportomedix.com) or call +49 (0)8251 901 469.

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